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Tel. +49 7463 99307 0
Fax: +49 7463 99307 33
E-Mail: mail@mondeal.de

Instruction for use for the MONDEAL BONE MILL (MBM)

Basic information

Basic instructions for the use of MONDEAL Medical Systems GmbH (hereinafter referred to as MONDEAL) implants and instruments in the fields of orthopedics and osteosynthesis.

It is imperative that all explanations and instructions in this manual are observed and complied with. Should any uncertainties, discrepancies or questions arise, please therefore contact us before the products are used or prepared (again).

Information and Symbols on Labels

REF	Item number		Manufacturer
QTY	Specification of quantities in the packages		Date of manufacture
LOT	Batch code		Caution
	Non sterile		Observe Instructions for use
	Marking for medical devices in Risk class I		

Products

These instructions for use are valid for the "Mondeal Bone Mill" (MBM). All components are reusable. The spring-loaded milling mechanism ensures even pressure and minimal effort when turning up to a filling height of max. 1.5 cm.

Blade cylinder

Item number	Description	Size
33-10011	MBM blade cylinder size S, small	approx. 2.0 mm
33-10021	MBM blade cylinder size M, medium	approx. 3.0 mm
33-10031	MBM blade cylinder size L, big	approx. 4.0 mm

The MBM body is made of aluminium with an abrasion- and heat-resistant surface seal. The pressed in ring for the running surface of the blade cylinder and the blocking blocks are made of biocompatible abrasion-resistant plastic. All other components are manufactured from stainless steel (in accordance with ISO 7153-1).

All components of the MBM Bone Mill are placed on the market **NON-STERILE** and must undergo an appropriate preparation process prior to first use.

Intended use

The MONDEAL Bone Mill (MBM) and accessories are intended for the manual processing of autologous bone grafts made from cancellous and/or compact bone. Bone grafts from the following regions are used for this purpose, depending on the choice of the attending physician: calotte bone, jaw bone, pelvic bone, etc.

Possible complications

- Complications that may possibly arise can be caused by
- Inadequate sterilization which does not accord with the guidelines. This may cause the material of the base body to be severely attacked and can lead to discoloration and contact with the graft material; furthermore, the locking mechanism of the pivot lever may stop functioning.
 - Insufficient or omitted manual pre-cleaning causes the bone material to be contaminated with non-sterile bone material from previous applications. This can lead to infections and to treatments using milled bone grafts failing.
 - Corrosion as a result of faulty positioning during ultrasound, in the cleaning and disinfection unit or autoclave due to residual fluid in the MBM.
 - Incorrect insertion and use of the blocking blocks may cause the MBM to become blocked and make milling impossible.
 - Blunt or damaged blade cylinders cause disproportionate, high wear on the MBM's running surfaces for the blade cylinder.
 - Jamming of the press punch due to lack of mobility in the upper MBM body, caused by lack of maintenance using an appropriate instrument care oil.
 - Graft pieces that are too small can lead to jamming if the wrong blade cylinder size is used. Irritation to soft tissue.

Warnings and precautionary measures

- As the manufacturing company, MONDEAL recommends the user read all available documents thoroughly before the first use in practice, and contact users who already have experience with this type of treatment method
- The products mentioned may only be used by specialist medical staff with the appropriate training.
- The product must be handled and stored with care. Damage to or scratches on the instrument may affect the safety of the product.
- Products damaged by transport, handling in the clinic or other causes may not be used under any circumstances!
- Use the necessary care when using and storing the products.

Use of original products

All system components have been designed and manufactured for a specific purpose and are therefore finely harmonized with one another. No component may be modified by the user or replaced by any instrument or product from a third-party manufacturer, even if it resembles or even exactly matches the size or shape of the original product. Insufficient quantity or quality of bone mass/substance for stable fixation.

Components of the MBM

The MBM bone mill consists of:

- MBM bone mill body
- Pivot lever
- Blade cylinder
- Bone collecting container for MBM
- Blocking blocks for MBM

Returns

Any product return shipments may only be sent back to us after disinfection/sterilization has been performed which is clearly evident (appropriate packaging with sterility indicators, decontamination certificate, etc.) The relevant ordinance on hygiene and commercial premises must be adhered to.

Guarantee

MONDEAL delivers only tested and fault-free products to its customers. All our products are designed and manufactured to meet the highest quality standards. MONDEAL, as the manufacturer of the products, excludes all warranty claims, and assumes no liability for direct or consequential damages, resulting from:

- Unintended use.
- Improper use, application or handling.
- Improper preparation and sterilization.
- Not observing the instructions for use.

MBM assembly instructions

- 1** Insertion of the blade cylinder.
Completely insert the rotary cylinder.
- Attach the pivot lever to the shaft of the blade cylinder
 1. Open the locking mechanism.
 2. Slide the lever onto the shaft as far as it will go.
 3. Close the locking mechanism.
- Remove the pivot lever from the shaft of the blade cylinder
 1. Open the locking mechanism.
 2. Slide the lever off the shaft.
- 3** Screw the bone collecting container into the blade cylinder
- 4** The MBM is ready to use
- 5** Insert the blocking blocks
(Might be necessary depending on the bone graft)
- 6** Spring-loaded mechanism is blocked due to the blocking blocks
- After the milling process
 1. Place the MBM on the bone collecting container. Do not release the bone mill
 2. Remove the pivot lever.
 3. Pull the base body of the MBM upwards.
- Remove the bone material
 1. Tap the collecting container on its base one or two times with force.
 2. Now separate the two parts and take out the bones.

MBM disassembly instructions

Disassembly of the bone mill is performed in the reverse sequence of the MBM assembly instructions

Never loosen let alone open any other screws or connecting parts on the MBM bone mill. This would impair functionality, and the guarantee of the MBM bone mill would be voided.

Notes on cleaning, disinfection and sterilization

All implants, instruments and containers of MONDEAL systems are delivered NON-STERILE and must be cleaned, disinfected and sterilized before each use; this also applies to the first use after delivery (after removal of the protective packaging for transport). Effective cleaning and disinfection are essential requirements for effective sterilization.

If possible, a machine method should be used for cleaning and disinfection. Manual methods – even using an ultrasonic bath – should not be used as these are significantly less effective and may cause damage.

For a description of the validated procedure for cleaning, disinfection and sterilization, see

"Preparation instructions, cleaning, disinfection and sterilization"

Additional information

You can request further information on the products (e.g. surgical techniques, care, cleaning, disinfection and sterilization) from your contact person. In addition, you will find all the information on the Internet at www.mondeal.de

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