MBM

MONDEAL BONE MILL

Gebrauchsanweisung	DE	Deutsch	Deutsch	Seite	2
Instruction for use	EN	Englisch	English	Page	3
Manual de instrucciones	ES	Spanisch	Español	Página	4
Manuel d'instructions	FR	Französisch	Français	Page	5
Инструкции за използване	BG	Bulgarisch	Български	Страница	6
Návod k použití	CZ	Tschechisch	Čeština	Strana	7
Brugsanvisning	DK	Dänisch	Dansk	Side	8
Kasutusjuhend	EE	Estisch	Eesti	Lehekülg	9
Οδηγίες χρήσης	EL	Griechisch	Ελληνικά	Σελίδα	10
Käyttöohjeet	FI	Finnisch	Suomi	Sivu	11
Upute za uporabu	HR	Kroatisch	Hrvatski	Stranica	12
Használati útmutató	HU	Ungarisch	Magyar	Oldal	13
Istruzioni d'uso	IT	Italienisch	Italiano	Pagina	14
Naudojimo instrukcija	LT	Litauisch	Lietuvių	Puslapis	15
Lietošanas instrukcija	LV	Lettisch	Latviešu	Lappuse	16
Bruksanvisning	NB	Norwegisch	Norsk	Side	17
Gebruiksaanwijzing	NL	Niederländisch	Nederlands	Pagina	18
Instrukcja użytkowania	PL	Polnisch	Polski	Strona	19
Instruções de utilização	PT	Portugiesisch	Português	Página	20
Instrucțiuni de utilizare	RO	Rumänisch	Română	Pagina	21
Návod na použitie	SK	Slowakisch	Slovenský	Strana	22
Navodila za uporabo	SL	Slowenisch	Slovenščina	Stran	23
Bruksanvisning	SV	Schwedisch	Svenska	Sidan	24

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Tel. +49 7463 99307 0
Fax: +49 7463 99307 33
E-Mail: mail@mondeal.de

Instruction for use for the MONDEAL BONE MILL (MBM)

Basic information

Basic instructions for the use of MONDEAL Medical Systems GmbH (hereinafter referred to as MONDEAL) implants and instruments in the fields of orthopedics and osteosynthesis.

It is imperative that all explanations and instructions in this manual are observed and complied with. Should any uncertainties, discrepancies or questions arise, please therefore contact us before the products are used or prepared (again).

Information and Symbols on Labels

 REF	Item number		Manufacturer
 QTY	Quantities		Date of manufacture
 LOT	Batch code		Caution
	Non sterile		Consult instructions for use
	Marking for medical devices in Risk class I		

Products

These instructions for use are valid for the "Mondeal Bone Mill" (MBM). All components are reusable. The spring-loaded milling mechanism ensures even pressure and minimal effort when turning up to a filling height of max. 1.5 cm.

Blade cylinder

Item number	Description	Size
33-10011	MBM blade cylinder size S, small	approx. 2.0 mm
33-10021	MBM blade cylinder size M, medium	approx. 3.0 mm
33-10031	MBM blade cylinder size L, big	approx. 4.0 mm

The MBM body is made of aluminium with an abrasion- and heat-resistant surface seal. The pressed in ring for the running surface of the blade cylinder and the blocking blocks are made of biocompatible abrasion-resistant plastic. All other components are manufactured from stainless steel (in accordance with ISO 7153-1).

All components of the MBM Bone Mill are placed on the market **NON-STERILE** and must undergo an appropriate preparation process prior to first use.

Intended use

The MONDEAL Bone Mill (MBM) and accessories are intended for the manual processing of autologous bone grafts made from cancellous and/or compact bone. Bone grafts from the following regions are used for this purpose, depending on the choice of the attending physician: calotte bone, jaw bone, pelvic bone, etc.

Possible complications

Complications that may possibly arise can be caused by

- Inadequate cleaning and sterilization that has not been carried out in accordance with the specifications. This can lead to discoloration of the surface as well as a limited function of the mechanical parts.
- Insufficient or omitted manual pre-cleaning causes the bone material to be contaminated with non-sterile bone material from previous applications. This can lead to infections and to treatments using milled bone grafts failing.
- Corrosion as a result of faulty positioning during ultrasound, in the cleaning and disinfection unit or autoclave due to residual fluid in the MBM.
- Incorrect insertion and use of the blocking blocks may cause the MBM to become blocked and make milling impossible.
- Blunt or damaged blade cylinders cause disproportionate, high wear on the MBM's running surfaces for the blade cylinder.
- Jamming of the press punch due to lack of mobility in the upper MBM body, caused by lack of maintenance using an appropriate instrument care oil.
- Graft pieces that are too small can lead to jamming if the wrong blade cylinder size is used. Irritation to soft tissue.

Warnings and precautionary measures

- As the manufacturing company, MONDEAL recommends the user read all available documents thoroughly before the first use in practice, and contact users who already have experience with this type of treatment method
- The products mentioned may only be used by specialist medical staff with the appropriate training.
- The product must be handled and stored with care. Damage to or scratches on the instrument may affect the safety of the product.
- Products damaged by transport, handling in the clinic or other causes may not be used under any circumstances!
- Use the necessary care when using and storing the products.

Use of original products

All system components have been designed and manufactured for a specific purpose and are therefore finely harmonized with one another. No component may be modified by the user or replaced by any instrument or product from a third-party manufacturer, even if it resembles or even exactly matches the size or shape of the original product. Insufficient quantity or quality of bone mass/substance for stable fixation.

Components of the MBM

The MBM bone mill consists of:

- MBM Bone Mill basic body
- MBM Turning lever
- MBM Cutting cylinder
- Bone collector for MBM
- Locking pieces for MBM

Returns

Any product return shipments may only be sent back to us after disinfection/sterilization has been performed which is clearly evident (appropriate packaging with sterility indicators, decontamination certificate, etc.) The relevant ordinance on hygiene and commercial premises must be adhered to.

Guarantee

MONDEAL delivers only tested and fault-free products to its customers. All our products are designed and manufactured to meet the highest quality standards. MONDEAL, as the manufacturer of the products, excludes all warranty claims, and assumes no liability for direct or consequential damages, resulting from:

- Unintended use.
- Improper use, application or handling.
- Improper preparation and sterilization.
- Not observing the instructions for use.

MBM assembly instructions

Insertion of the blade cylinder. Completely insert the rotary cylinder.

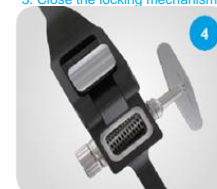
Attach turning lever to the shaft of the blade cylinder
1. Open the locking mechanism.
2. Slide the lever onto the shaft as far as it will go.
3. Close the locking mechanism.

Remove the turning lever from the shaft of the blade cylinder
1. Open the locking mechanism.
2. Slide the lever off the shaft.


Screw the bone collector into the blade cylinder.

The MBM is ready to use.

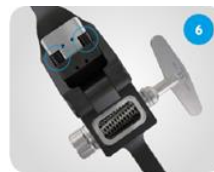Insert the locking pieces (might be necessary depending on the bone graft).

Spring-loaded mechanism is blocked due to the blocking blocks.

After the milling process
1. Place the MBM on the bone collector. Do not release the bone mill.
2. Remove the turning lever.
3. Pull the base body of the MBM upwards.

Remove the bone material
1. Tap the collecting container on its base one or two times with force.
2. Now separate the two parts and take out the bones.

MBM disassembly instructions

Disassembly of the bone mill is performed in the reverse sequence of the MBM assembly instructions

Never loosen let alone open any other screws or connecting parts on the MBM bone mill. This would impair functionality, and the guarantee of the MBM bone mill would be voided.

Notes on cleaning, disinfection and sterilization

All implants, instruments and containers of MONDEAL systems that have been delivered **NON-STERILE** must be cleaned, disinfected and sterilized before each use; this also applies to the first use after delivery (after removal of the protective packaging for transport). Effective cleaning and disinfection are essential requirements for effective sterilization.

A mechanical process must be used for cleaning and disinfection.

For a description of the validated procedure for cleaning, disinfection and sterilization, see

"Preparation instructions, cleaning, disinfection and sterilization"

Additional information

You can request further information on the products (e.g. surgical techniques, care, cleaning, disinfection and sterilization) from your contact person. In addition, you will find all the information on the Internet at www.mondeal.de

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